

PRODUCT INFORMATION DATA SHEET



The following information should be read before using this device.

REF xSE series

◆ Description:

The Disposable Smoke Evacuation Tubing consists of one smoke filter and tubing, which can be connected to the stopcock on the trocar.



◆ Intended Use:

The Disposable Smoke Evacuation Tubing is used as a smoke filter for the evacuation of toxic surgical smoke during laparoscopic procedure.

◆ Note:

It is supplied sterile unless package is damaged or opened. All contents are ETO sterilized.

◆ Instructions for Use:

1. Peel the pouch to access sterile product.
2. Place Disposable Smoke Evacuation Tubing on sterile field.
3. Attach luer lock connector in the distal end of the tubing to trocar.
4. To remove the smoke, make sure the flap on the stopcock is in vertical position to initiate the evacuation of smoke.

⚠ Warnings and Precautions:

- Federal (U.S.A) law restricts this device to sale, distribution, and use by, or on the order of a physician.
- This device is EO sterilized and for single use only. Please do not re-use or re-process to avoid irreversible injury such as cross-infection or injury caused by device malfunction.
- Do not use when the package is opened or damaged.
- Store this device in a dry, clean and safe place. Keep the device away from flammable and combustible chemicals or materials.
- Please follow universal guidelines upon medical waste disposal to avoid any potential cross-infection, environmental contamination or injury caused by sharps waste.



This Product Information Data Sheet is subject to change without notice.

Version: 8

Revision Date: 2015/03/30

4MSE04



UNIMAX MEDICAL SYSTEMS INC

8F-2, NO. 127, Lane 235, Pao Chiao Rd, Hsin Tien city, Taipei, Taiwan.

Tel: +886-2-8919-1698 Fax: +886-2-8919-1528

www.unimaxmeds.com , Email:unimax@unimaxmeds.com



MDI Europa GmbH

Langenhagener Str. 71, 30855 Langenhagen, Germany

Tel: +49-511-3908 9530

Rx only



0434

PRODUCT INFORMATION DATA SHEET